Specifications

- Designed to provide up to 50+ decibels of ambient sound amplification. Maximum output of 107dB.
- Frequency response: 100 to 14000 hertz.
- Solid state microelectronic circuitry requires only two AAA (1.5V) alkaline batteries for up to 80 hours of operation. Batteries are included.
- Premium stereo earphones deliver clear, crisp sound particularly well in the human voice range.
- Operating temperature range 0°C min, 55°C max.
- Relative humidity operating range: 20°C/95%RH

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5 YEAR WARRANTY

Sonic Technology Products, Inc. warrants to the original purchaser that your SuperEarPlus® will be free of defects in materials or workmanship for a period of 5 years from the date of purchase. Warranty is void if defects are caused by negligence, abuse, misuse or damage caused by water or heat. Returns made under this Warranty must be accompanied by \$3.00 to cover the handling and return postage cost. This Warranty gives you specific legal rights and you may have other rights which may vary from state to state. Should it be necessary to send your SuperEarPlus® in for service under the terms of this Warranty, return it in a well-packaged carton and ship postage prepaid to the address below.

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SONIC TECHNOLOGY PRODUCTS, INC.
120 Richardson St., Grass Valley, CA 95945 USA

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SuperEar **Plus**[®]
PERSONAL
SOUND
AMPLIFIER

Model SE7500

SuperEarPlus Owner's Manual
Manuel de l'utilisateur du SuperEarPlus
SuperEarPlus Manual de Consumidor
SuperEarPlus Handbuch Des Inhabers

Instructions for Use

1. Remove the battery compartment cover by lightly depressing the cover with your thumb and slide it in the direction of the arrow.

2. Insert the two AAA batteries. Be sure to match the positive (+) and negative (-) ends to those indicated on the battery cavity. Alkaline batteries will deliver 2-3 times the operational life of lesser batteries.

3. Slide battery compartment cover back into locked position.

4. Insert the headphone/earbud connector into the jack on the left side of the amplifier.

5. Be sure the volume control is in the "low" position before putting on the headphones/earbuds and adjusting for comfort.

6. Depress the "on" button and slowly turn the volume control knob to the desired level of amplification.

7. The red LED light will illuminate when the SuperEarPlus is turned on.

8. Unless the volume is adjusted, the SuperEarPlus will automatically turn itself off after 30 minutes to conserve battery life. To turn on again, depress the on/off button.

9. This amplifier has a maximum gain of over 50 dB. At approximately 1/4 open on the volume control, there will be about a 10dB sound gain. The maximum gain of well over 50 dB is

delivered with the volume wheel all the way open. You may not need full volume gain. Choose an amplification level that is most comfortable for you.

10. If headphones/earbuds not in use are placed near the amplifier while it is turned on, feedback squeal will result. **11. DO NOT REMOVE THE UNIT FROM THE CARRYING CASE BY GRASPING THE MICROPHONE.** This may cause the microphone to detach from the unit. Instead, PUSH the unit out through the hole in the bottom of the case.

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©Sonic Technology 2010